



SPORE TESTING

Ensuring Proper Sterilization of Instruments.

P *Always follow manufacturer's written instructions. Refer to policy

1. START HERE



CLEAN*

Soak, remove debris, rinse instruments & inspect (never sterilize dirty instrument)

2.

PACKAGE & LOAD

Write on the pouches: Operators initials date of sterilization load run identification information place pouches on its side (preferably on a rack) with space in between.

3.

STERILIZE

Instruments must be exposed to steam at the correct:

TIME TEMP PRESSURE



*Sterilizer parameters



5.3 **P**

Inspect & repair autoclave Repeat spore test a total of **3 TIMES** document results (*Refer to policy for continued failed spore tests)

***No** Pass spore test?

5.2

Yes

Resterilize previously recalled instruments.

5.1

NON-PASSING SPORE TESTS:

- ☐ Remove autoclave from service
- ☐ Report to supervisor
- ☐ Recall all instruments
- ☐ Re-spore test
- ☐ Document results

P



4.

CHEMICAL QUALITY INDICATOR

(i.e. heat sensitive tape)

1. Correct processing:



Chemical indicators **change color** when processed correctly.

P

2. Document results

6.

CLEANING & MAINTENANCE

- Follow daily, weekly & monthly cleaning per manufacturing guidelines
- Complete annual maintenance



STORAGE

Protect packages & keep in designated pouch. Place newer pouches in back Resterilize pouches only when:

- opened
- wet/moist
- discolored/damaged

Routinely check status of packages (e.g. assign 1 person to keep track) Calendar routine checks

Yes

Pass spore test?

No

5.

SPORE TEST

Complete spore testing. Hold instruments until spore test results are received.

END HERE

HEALTHY PEOPLE. HEALTHY COMMUNITIES.

www.thealliance.health

11-2021